



Letter of Confirmation

Date : Feb 20, 2020

Total : 1 page (Including this page)

To whom it may concern,

BT Inc, incorporated in South Korea with its principle office and manufacturing facility at A-313, Samsung Techono-valley, 140, Tongil-ro, Deogyang-gu, Goyang-si, Gyeonggi-do, 10594, Korea, hereby a confirms alcohol-based wipes can be used on BT Inc's CPR Manikins including Sherpa and Sherpa X.

Yours Sincerely,

TK

Managing Director

BT Inc Seoul

A handwritten signature in blue ink that reads 'TK Kim'.

ENF KOREA CO
(BT Inc Seoul)

A handwritten signature in blue ink that reads 'TK Kim'.


MANAGING DIRECTOR

How to Manage the Airway and Lung Clean

1. Remove the chest skin.



2. Disconnect the white airway part from the manikin head and separate the T-shaped connector from the manikin's neck.

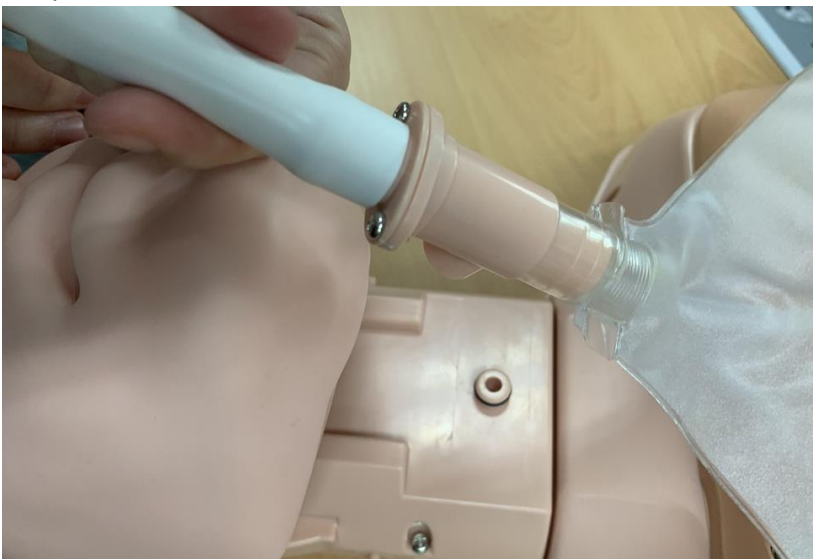

3. Loosen screws from the connector.



4. You can disconnect a lung to replace it.
To separate the white part from the connector, pull it out.
You can rinse, sanitize these 2 plastic parts with water or alcohol spray.



*Make sure to dry the part before assemble it.

5. Reassemble separated parts to the manikin before use it.